





# GE Medical Systems

## GE HISPEED ADVANTAGE RP CT SCANNER

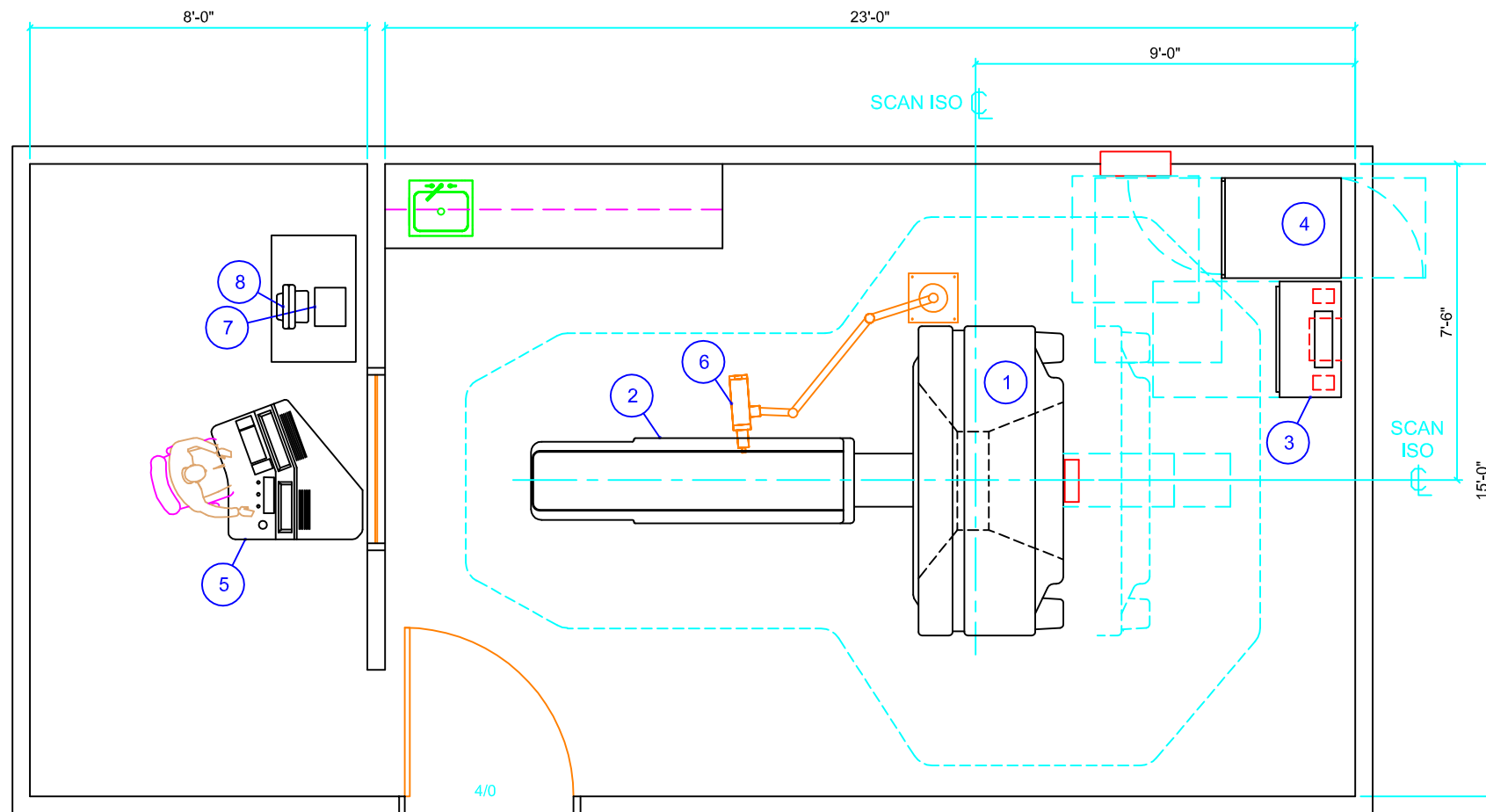

**TYPICAL ROOM AND EQUIPMENT LAYOUT**  
(DASHED LINES DEPICT EQUIPMENT TRAVEL AND SERVICE CLEARANCES)  
MINIMUM FINISHED CEILING HEIGHT = 9'-0"

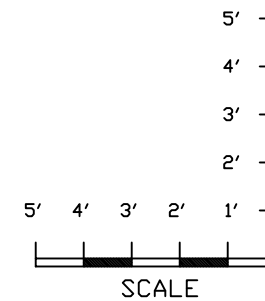

### GENERAL NOTES

- ALL SPECIFICATIONS AND SPACE REQUIREMENTS ARE SHOWN GENERICALLY. CONTACT GAMMA TECH, INC. FOR SPECIFIC REQUIREMENTS PERTAINING TO OPTIONAL EQUIPMENT COMPONENTS AND ADDITIONAL ROOM PREPARATION SPECIFICATIONS.
- MINIMUM FINISHED CEILING HEIGHT IS 9'-0".
- AMBIENT OPERATION TEMPERATURE: 66° TO 75° F.  
HUMIDITY: 30% TO 60% (NON-CONDENSING).
- MAIN INCOMING POWER SUPPLY:  
480VAC, THREE PHASE, 112.5KVA, DELTA OR WYE, 125AMP, 3 POLE, MAIN CB

© PLANNING DIVISION

FILE NAME:  
GE HISPEED ADVANTAGE  
RP

REVISION DATE:  
01.24.2008

### EQUIPMENT SCHEDULE

#### EQUIPMENT IDENTIFICATION NUMBER

EQUIPMENT IDENTIFICATION NUMBER	BTU / HOUR	DESCRIPTION	WEIGHT (LBS.)			
			DIMENSIONS (IN.)			
			W	x	D	x
1	24400	GENERAL ELECTRIC MEDICAL SYSTEMS HISPEED ADVANTAGE RP CT SCANNER GANTRY	88.00	42.75	78.55	3700
2	700	GENERAL ELECTRIC MEDICAL SYSTEMS HISPEED ADVANTAGE RP CT SCANNER PATIENT TABLE	24.25	91.91	43.78	1185
3	3400	GENERAL ELECTRIC MEDICAL SYSTEMS HISPEED ADVANTAGE RP CT SCANNER POWER DISTRIBUTION UNIT	33.00	18.50	78.00	1088
4	6800	GENERAL ELECTRIC MEDICAL SYSTEMS HISPEED ADVANTAGE RP CT SCANNER SCAN CONTROL UNIT	28.50	34.00	70.25	800
5	2400	GENERAL ELECTRIC MEDICAL SYSTEMS HISPEED ADVANTAGE RP CT SCANNER OPERATOR'S CONSOLE	39.70	43.56	48.53	270
6	N/A	MEDRAD STELLANT CT INJECTOR SYSTEM HEAD ON CEILING MOUNT COUNTERPOISE				
7	N/A	MEDRAD STELLANT CT INJECTOR SYSTEM ELECTRONICS BASE UNIT (ON DESK)	11.00	8.80	11.50	14
8	N/A	MEDRAD STELLANT CT INJECTOR SYSTEM DISPLAY CONTROL CONSOLE (ON DESK)	12.50	8.90	13.50	8

PLANNING BY:



**GAMMA TECH, INC**

129 Wild Berry Lane  
Hampstead, NC 28443  
Ph (910) 270-9787 • Fax (910) 270-9767  
www.gammatech.net