



GE Medical Systems

GE HISPEED ADVANTAGE Z CT SCANNER


GENERAL NOTES

- ALL SPECIFICATIONS AND SPACE REQUIREMENTS ARE SHOWN GENERALLY. CONTACT GAMMA TECH, INC. FOR SPECIFIC REQUIREMENTS PERTAINING TO OPTIONAL EQUIPMENT COMPONENTS AND ADDITIONAL ROOM PREPARATION SPECIFICATIONS.
- MINIMUM FINISHED CEILING HEIGHT IS 9'-0".
- AMBIENT OPERATION TEMPERATURE: 66° TO 75° F.
HUMIDITY: 30% TO 60% (NON-CONDENSING).
- MAIN INCOMING POWER SUPPLY:
480VAC, THREE PHASE, 112.5KVA, DELTA OR WYE, 125AMP, 3 POLE, MAIN CB

© PLANNING DIVISION

FILE NAME:
GE HISPEED ADVANTAGE Z

REVISION DATE:
01.24.2008

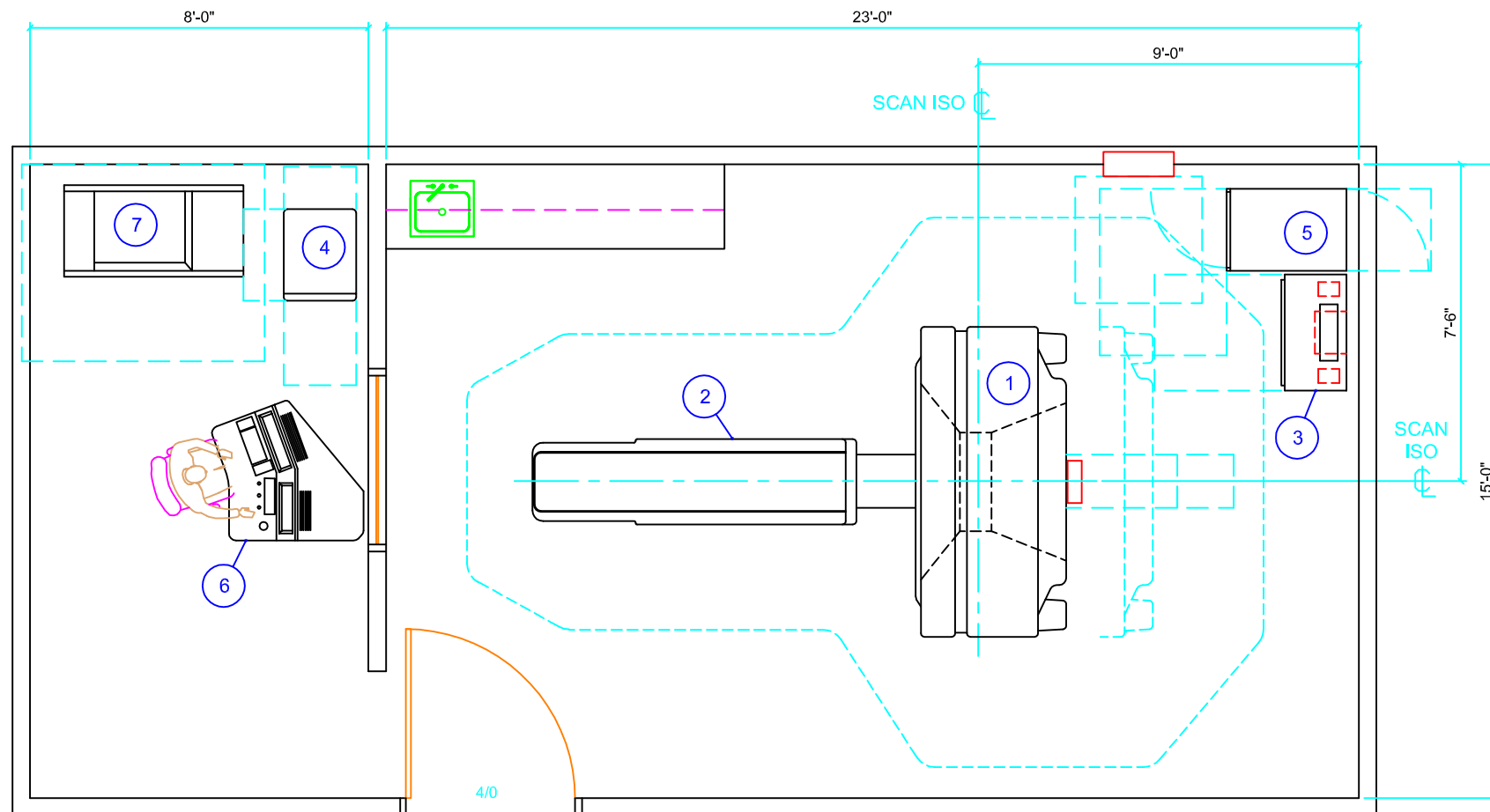

TYPICAL ROOM AND EQUIPMENT LAYOUT
(DASHED LINES DEPICT EQUIPMENT TRAVEL AND SERVICE CLEARANCES)
MINIMUM FINISHED CEILING HEIGHT = 9'-0"

EQUIPMENT SCHEDULE

EQUIPMENT IDENTIFICATION NUMBER

EQUIPMENT IDENTIFICATION NUMBER	BTU / HOUR	DESCRIPTION	WEIGHT (LBS.)			
			DIMENSIONS (IN.)			
			W	x	D	x
1	24400	GENERAL ELECTRIC MEDICAL SYSTEMS HISPEED ADVANTAGE Z CT SCANNER GANTRY	88.00	42.75	78.55	3700
2	700	GENERAL ELECTRIC MEDICAL SYSTEMS HISPEED ADVANTAGE Z CT SCANNER PATIENT TABLE	24.25	91.91	43.78	1185
3	3400	GENERAL ELECTRIC MEDICAL SYSTEMS HISPEED ADVANTAGE Z CT SCANNER POWER DISTRIBUTION UNIT	33.00	18.50	78.00	1088
4	5450	GENERAL ELECTRIC MEDICAL SYSTEMS HISPEED ADVANTAGE Z CT SCANNER COMPUTER CABINET	19.50	25.20	26.50	250
5	3400	GENERAL ELECTRIC MEDICAL SYSTEMS HISPEED ADVANTAGE Z CT SCANNER SCAN CONTROL UNIT	23.30	34.00	70.25	500
6	2400	GENERAL ELECTRIC MEDICAL SYSTEMS HISPEED ADVANTAGE Z CT SCANNER OPERATORS CONSOLE	39.70	43.56	48.53	270
7	2220	(OPTIONAL) 3M M-952 LASER IMAGER CAMERA	50.79	25.59	47.24	688

PLANNING BY:



GAMMA TECH, INC

129 Wild Berry Lane
Hampstead, NC 28443
Ph (910) 270-9787 • Fax (910) 270-9767
www.gammatech.net

