



# GE Medical Systems

## GE HISPEED ADVANTAGE CT SCANNER

### GENERAL NOTES

- ALL SPECIFICATIONS AND SPACE REQUIREMENTS ARE SHOWN GENERICALLY. CONTACT GAMMA TECH, INC. FOR SPECIFIC REQUIREMENTS PERTAINING TO OPTIONAL EQUIPMENT COMPONENTS AND ADDITIONAL ROOM PREPARATION SPECIFICATIONS.
- RECOMMENDED FINISHED CEILING HEIGHT IS 9'-0".
- AMBIENT OPERATION TEMPERATURE: 55° TO 75° F.  
HUMIDITY: 20% TO 80% (NON-CONDENSING).  
ALLOWABLE TEMPERATURE CHANGE: 15°F PER HOUR  
ALLOWABLE HUMIDITY CHANGE: 10% PER HOUR  
MAX - ALTITUDE: 8000FT ABOVE SEA LEVEL
- MAIN INCOMING POWER SUPPLY:  
480VAC, THREE PHASE, 112.5KVA, WYE, 125AMP, 3 POLE, MAIN CB  
5% VOLTAGE REGULATION, 85% POWER FACTOR

© PLANNING DIVISION


FILE NAME:  
GE HISPEED ADVANTAGE

REVISION DATE:  
01.22.2008

### EQUIPMENT SCHEDULE

#### EQUIPMENT IDENTIFICATION NUMBER

		BTU / HOUR	DESCRIPTION	WEIGHT (LBS.)	
				DIMENSIONS (IN.) W x D x H	
1	24400		GE MEDICAL SYSTEMS CT HISPEED ADVANTAGE GANTRY		3700
2	700		GE MEDICAL SYSTEMS PATIENT TABLE W/ EXTENDED TABLE TOP		1185
3	2400		GE MEDICAL SYSTEMS OPERATOR'S CONSOLE		270
4	3400		GE MEDICAL SYSTEMS POWER DISTRIBUTION UNIT		1088
5	5450		GE MEDICAL SYSTEMS MAIN COMPUTER CABINET		250
6	3400		GE MEDICAL SYSTEMS SCAN UNIT CABINET		500
7	7862		GE MEDICAL SYSTEMS INDEPENDENT CONSOLE AND COMPUTER		520
8	2220		GE MEDICAL SYSTEMS LASER IMAGER SYSTEM		686

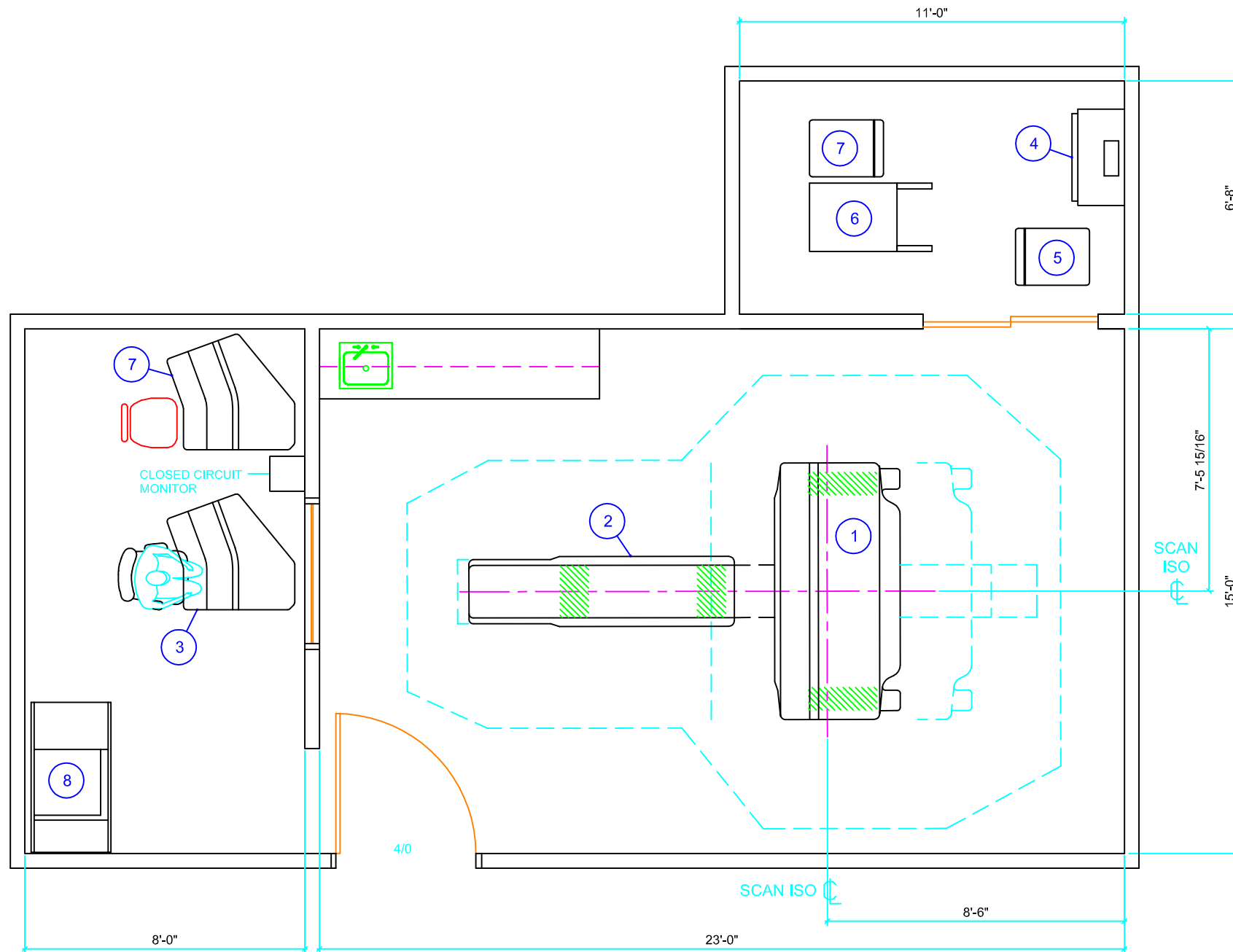

**TYPICAL ROOM AND EQUIPMENT LAYOUT**  
(DASHED LINES DEPICT EQUIPMENT TRAVEL AND SERVICE CLEARANCES)  
RECOMMENDED FINISHED CEILING HEIGHT = 9'-0"

PLANNING BY:



**GAMMA TECH, INC**

129 Wild Berry Lane  
Hampstead, NC 28443  
Ph (910) 270-9787 • Fax (910) 270-9767  
www.gammatech.net