


Neusoft®

NEUVIZ 64 CT SCANNER SUITE

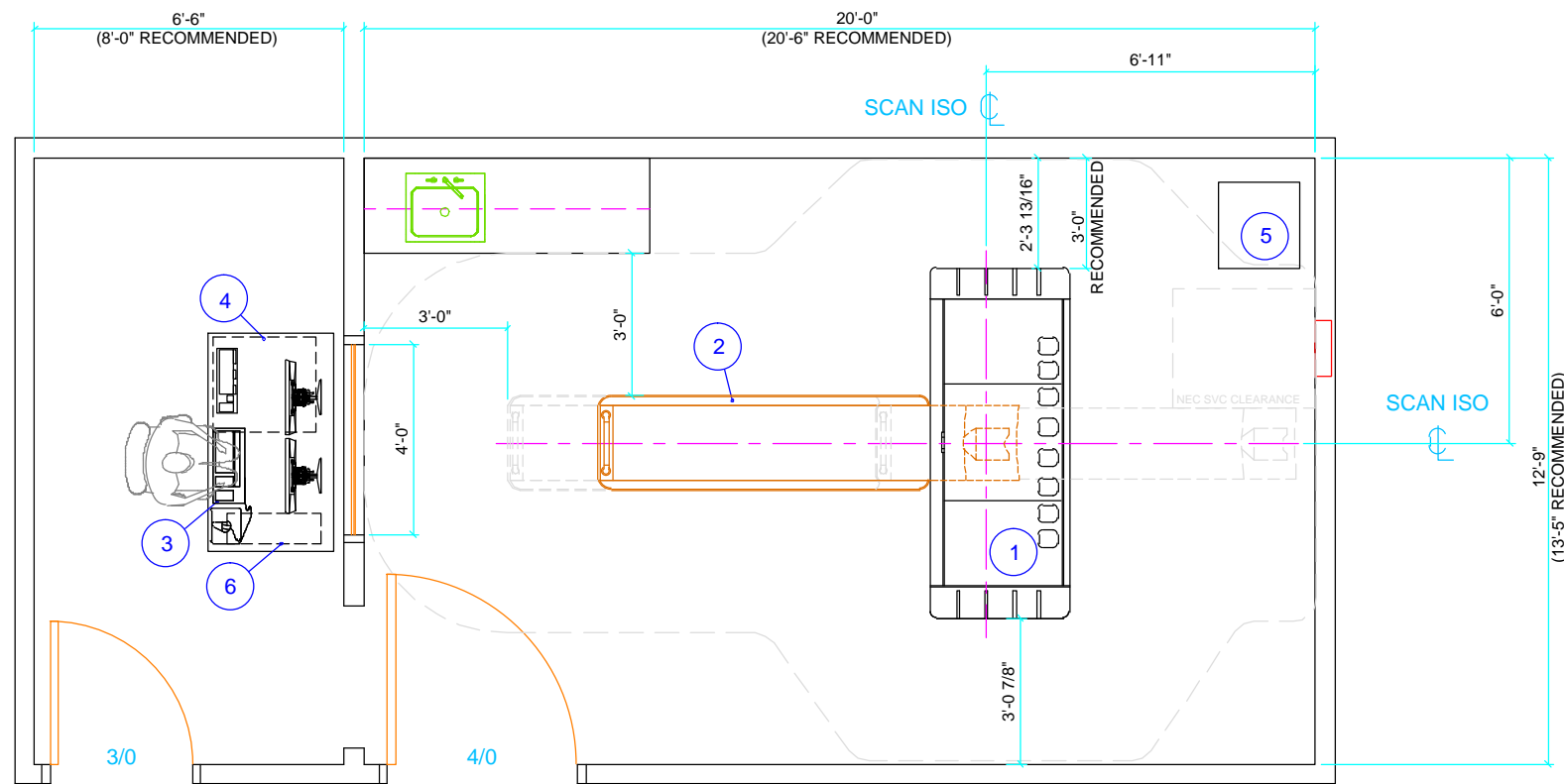


TYPICAL EQUIPMENT LAYOUT - MINIMUM ROOM SIZE

(DASHED LINES DEPICT EQUIPMENT TRAVEL AND SERVICE CLEARANCES)

(RECOMMENDED FINISHED CEILING HEIGHT = 9'-0")

(MINIMUM FINISHED CEILING HEIGHT = 8'-0")

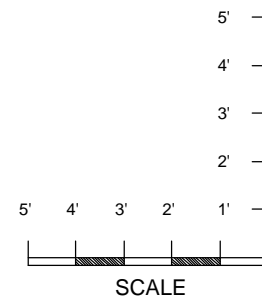

GENERAL NOTES

- ALL SPECIFICATIONS AND SPACE REQUIREMENTS ARE SHOWN GENERICALLY. CONTACT GAMMA TECH, INC. FOR SPECIFIC REQUIREMENTS PERTAINING TO OPTIONAL EQUIPMENT COMPONENTS AND ADDITIONAL ROOM PREPARATION SPECIFICATIONS.
- RECOMMENDED FINISHED CEILING HEIGHT IS 9'-0". MINIMUM FINISHED CEILING HEIGHT IS 8'-0".
- AMBIENT OPERATION TEMPERATURE: 64° F (18° C) TO 75° F (24° C). HUMIDITY: 30% TO 60% (NON-CONDENSING). ALLOWABLE TEMPERATURE CHANGE: 5° F (3° C) PER HOUR. ALLOWABLE HUMIDITY CHANGE: 5% PER HOUR
- MAIN INCOMING POWER SUPPLY:
 NEUVIZ 64i: 200 - 480VAC, THREE PHASE, 80KVA, 50/60HZ, DELTA OR WYE 100 AMP, 3 POLE (480V), 200 AMP, 3 POLE (208V), MAIN CB. ±5% VOLTAGE REGULATION.
 NEUVIZ 64e: 200 - 480VAC, THREE PHASE, 100KVA, 50/60HZ, DELTA OR WYE 150 AMP, 3 POLE (480V), 250 AMP, 3 POLE (208V), MAIN CB. ±5% VOLTAGE REGULATION.

© PLANNING DIVISION

FILE NAME:
NV64 CT

REVISION DATE:
06.16.2014

EQUIPMENT SCHEDULE

EQUIPMENT IDENTIFICATION NUMBER

ID	BTU / HOUR [W / HR]	DESCRIPTION	WEIGHT LBS. [KG]			
			DIMENSIONS IN. [MM]			
			W	x	D	x
1	17075 [5004]	NEUSOFT MEDICAL SYSTEMS NEUVIZ 64 SLICE CT GANTRY	88.34 [2244]	35.04 890	75.59 1920	4078 1800]
2		NEUSOFT MEDICAL SYSTEMS NEUVIZ 64 SLICE CT PATIENT TABLE	23.89 [607]	97.83 2485	43.31 1100	772 350]
3	2561 [751]	NEUSOFT MEDICAL SYSTEMS NEUVIZ 64 SLICE CT OPERATORS CONSOLE DESK	55.12 [1400]	31.49 800	29.92 760	102 46]
4		NEUSOFT MEDICAL SYSTEMS NEUVIZ 64 SLICE CT COMPUTER RACK CABINET (UNDER DESK/COUNTER)	24.01 [610]	25.98 660	28.93 735	77 35]
5	375 [110]	NEUSOFT MEDICAL SYSTEMS NEUVIZ 64 SLICE CT CONSOLE UPS (UNDER DESK/ COUNTER)	7.90 [200]	23.70 600	17.40 440	123 56]
6	1865 [547]	NEUSOFT MEDICAL SYSTEMS NEUVIZ 64 SLICE CT TEAL ISOLATION TRANSFORMER POWER UNIT	21.76 [553]	20.34 516	28.06 713	603 274]

PLANNING BY:



GAMMA TECH, INC

129 Wild Berry Lane
Hampstead, NC 28443
Ph (910) 270-9787 • Fax (910) 270-9767
www.gammatech.net