



TYPICAL ROOM AND EQUIPMENT LAYOUT
(DASHED LINES DEPICT TABLE TOP TRAVEL AND SFD CLEARANCE.)

EQUIPMENT SCHEDULE

EQUIPMENT IDENTIFICATION NUMBER		DESCRIPTION	WEIGHT (LBS.)			
BTU / HOUR	W		x	D	x	H
1	12	SHIMADZU UD150B-30 X-RAY CONTROL CONSOLE COUNTER MOUNTED	13.20	11.40	3.15	12
2	3095	SHIMADZU UD150B-30 X-RAY CONTROL CABINET	31.50	15.80	68.90	551
3	1200	SHIMADZU YSF-220 90/45 R&F DIAGNOSTIC TABLE	82.67	52.75	88.98	2756
4	3377	SHIMADZU AUXILIARY ELECTRONICS CABINET (CONTAINS THE ANCILLARIES)	21.25	25.50	76.13	473
5	2364	SHIMADZU CH30GX OVERHEAD TUBE SUPPORT FRONT MOUNT X-RAY TUBE	217.00 (MODIFIED)	78.80	108.00	600
6	300	SHIMADZU MOBILE TV MONITOR CART	22.00	28.00	44.00	100
7	5120	SHIMADZU DIGITEX PRO DIGITAL COMPUTER MOUNTED UNDER THE COUNTER	8.70	22.00	23.60	77
8	410	SHIMADZU 17" IMAGE DISPLAY MONITOR MOUNTED ON THE COUNTER	16.50	16.50	16.10	44
9	680	SHIMADZU 21" IMAGE DISPLAY MONITOR MOUNTED ON MONITOR CART	19.30	21.60	19.30	55
10	N/A	SHIMADZU DIGITEX PRO POWER DISTRIBUTION BLOCK MOUNTED UNDER THE COUNTER	15.90	3.40	6.50	5
11	800	SHIMADZU DIGITEX PRO CAMERA CONTROL MOUNTED ON SHELF UNDER THE COUNTER	14.20	17.80	5.90	25
12	6800	SHIMADZU SERIES ST, MODEL 1020 UPS MOUNTED UNDER THE COUNTER	7.00	18.00	13.00	81
13	200	SHIMADZU BS-2000 V VERTICAL BUCKY STAND.	31.61	16.73	87.79	300

GENERAL NOTES

- ALL SPECIFICATIONS AND SPACE REQUIREMENTS ARE SHOWN GENERICALLY. CONTACT GAMMA TECHNOLOGIES FOR SPECIFIC REQUIREMENTS PERTAINING TO OPTIONAL EQUIPMENT COMPONENTS AND ADDITIONAL ROOM PREPARATION SPECIFICATIONS.
- MINIMUM RECOMMENDED CEILING HEIGHT IS 9'-4".
- SUGGESTED AMBIENT ROOM REQUIREMENTS ARE 66° TO 75° F. AT 30% TO 60% R.M. (NON-CONDENSING).
- MAIN INCOMING POWER SUPPLY:
480VAC, THREE PHASE, 100KVA, DELTA OR WYE
5% VOLTAGE REGULATION, 95% POWER FACTOR
80AMP, 3 POLE, MAIN CB

SHIMADZU DIGITEX PRO RF SYSTEM

GAMMA TECH, INC



129 Wild Berry Lane
Hampstead, NC 28443
Ph (910) 270-9787
Fax (910) 270-9767
www.gammatech.net

© PLANNING DIV.

FILE NAME:
TYP YSF220
REVISION DATE:
04/15/2002