



# TOSHIBA

## AQUILION 16 SLICE CT SYSTEM

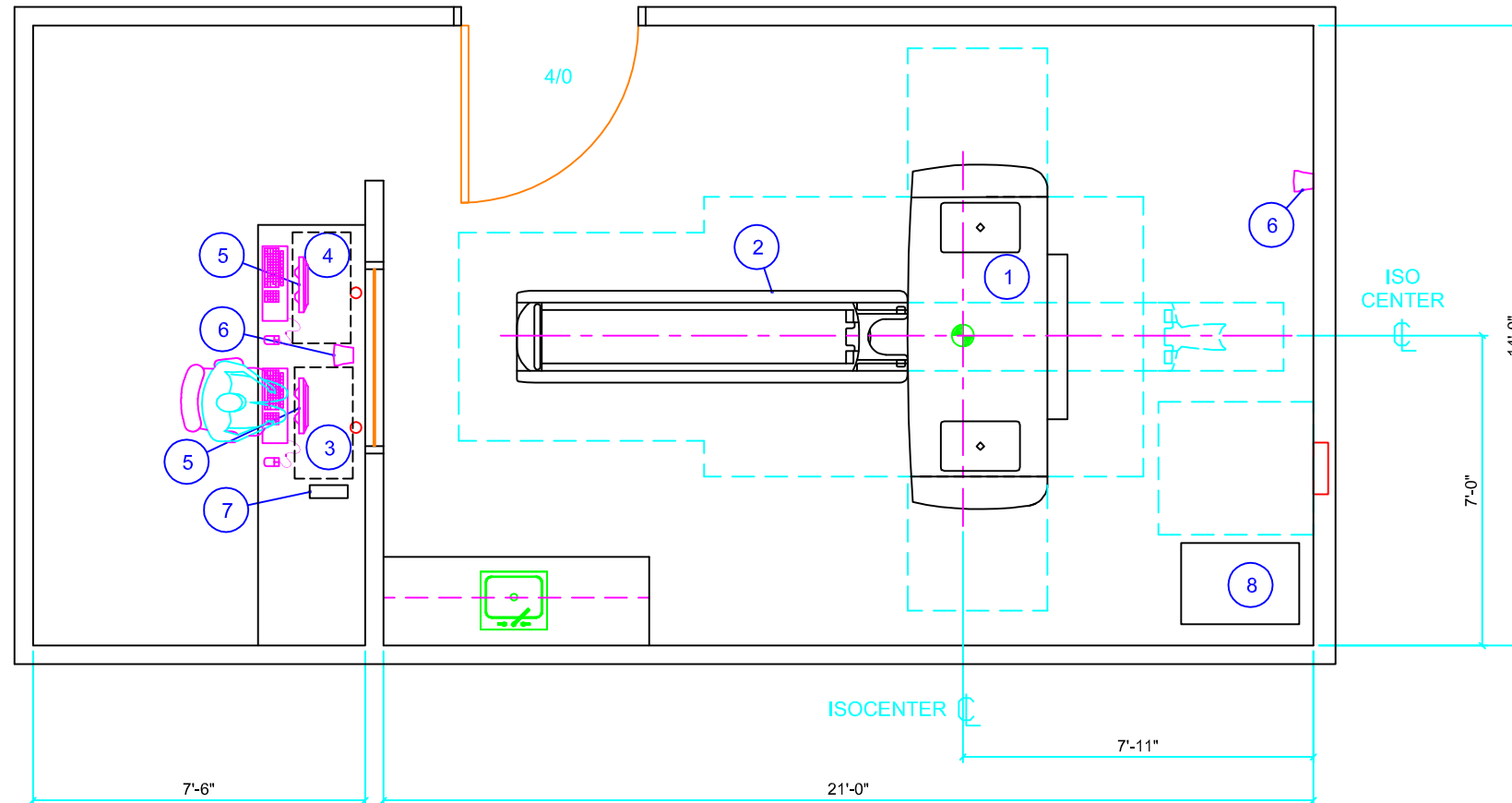

**TYPICAL EQUIPMENT LAYOUT**  
(DASHED LINES DEPICT EQUIPMENT TRAVEL AND SERVICE CLEARANCES)

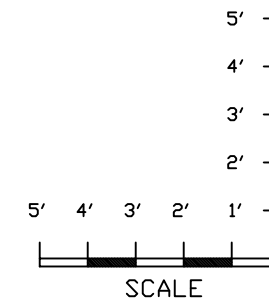

### GENERAL NOTES

- ALL SPECIFICATIONS AND SPACE REQUIREMENTS ARE SHOWN GENERICALLY. CONTACT GAMMA TECH, INC. FOR SPECIFIC REQUIREMENTS PERTAINING TO OPTIONAL EQUIPMENT COMPONENTS AND ADDITIONAL ROOM PREPARATION SPECIFICATIONS.
- RECOMMENDED FINISHED CEILING HEIGHT IS 9'-0". MINIMUM FINISHED CEILING HEIGHT IS 8'-2 1/2".
- AMBIENT OPERATION TEMPERATURE: 68° TO 74° F. HUMIDITY: 40% TO 70% (NON-CONDENSING). ALLOWABLE TEMPERATURE CHANGE: 15°F PER HOUR. ALLOWABLE HUMIDITY CHANGE: 10% PER HOUR.  
MAX - ALTITUDE: 8,000 FT. ABOVE SEA LEVEL
- MAIN INCOMING POWER SUPPLY:  
480VAC, THREE PHASE, 150KVA, DELTA OR WYE, 100AMP, 3 POLE, MAIN CB  
5% VOLTAGE REGULATION

© PLANNING DIVISION

FILE NAME:  
TOSHIBA AQUILION 16

REVISION DATE:  
01.06.2012

### EQUIPMENT SCHEDULE

#### EQUIPMENT IDENTIFICATION NUMBER

| ID | BTU / HOUR | DESCRIPTION  | WEIGHT (LBS.)    |        |        |      |
|----|------------|--|------------------|--------|--------|------|
|    |            |  | DIMENSIONS (IN.) |        |        |      |
|    |            |  | W                | x      | D      | x    |
| 1  | 34120      | TOSHIBA AQUILION 16 SLICE CT SCANNER GANTRY                                | 91.70            | 37.80  | 76.80  | 3859 |
| 2  | 1706       | TOSHIBA AQUILION 16 SLICE CT SCANNER PATIENT COUCH                         | 24.40            | 105.90 | VARIES | 993  |
|    |            | SHORT COUCH = 94.00  |                  |        |        |      |
| 3  | 4610       | TOSHIBA AQUILION 16 SLICE CT SCANNER PROCESSING UNIT (BENEATH COUNTER)     | 15.75            | 32.12  | 27.56  | 309  |
| 4  | 4610       | TOSHIBA AQUILION 16 SLICE CT SCANNER RECONSTRUCTION UNIT (BENEATH COUNTER) | 15.75            | 32.12  | 27.56  | 220  |
| 5  | 192        | TOSHIBA AQUILION 16 SLICE CT SCANNER PATIENT MONITOR (x2) (ON COUNTER)     | 15.75            | 8.00   | 16.00  | 18   |
| 6  | N/A        | TOSHIBA AQUILION 16 SLICE CT SCANNER SPEAKER (WALL/COUNTER MOUNTED) (x2)   | 5.50             | 5.31   | 7.88   | 5    |
| 7  | 455        | TOSHIBA AQUILION 16 SLICE CT SCANNER INNERSERVICE WORKSTATION (ON COUNTER) | 3.50             | 9.93   | 10.31  | 11   |
| 8  | 2000       | TOSHIBA PCDU-TW/U POWER CONDITIONING UNIT                                  | 32.00            | 22.00  | 56.00  | 710  |

PLANNING BY:



**GAMMA TECH, INC**

129 Wild Berry Lane  
Hampstead, NC 28443  
Ph (910) 270-9787 • Fax (910) 270-9767  
www.gammatech.net