


**TYPICAL ROOM AND EQUIPMENT LAYOUT**

(DASHED LINES DEPICT TABLE TOP TRAVEL)

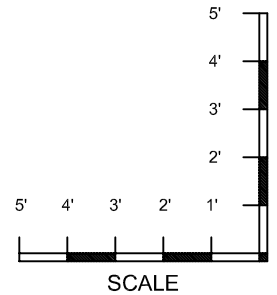

**GENERAL NOTES**

- ALL SPECIFICATIONS AND SPACE REQUIREMENTS ARE SHOWN GENERICALLY. CONTACT GAMMA TECHNOLOGIES FOR SPECIFIC REQUIREMENTS PERTAINING TO OPTIONAL EQUIPMENT COMPONENTS AND ADDITIONAL ROOM PREPARATION SPECIFICATIONS.
- MINIMUM RECOMMENDED CEILING HEIGHT IS 8'-0".
- SUGGESTED AMBIENT ROOM REQUIREMENTS ARE 68° TO 78° F. AT 40% TO 80% R.M. (NON-CONDENSING).
- MAIN INCOMING POWER SUPPLY:  
208VAC, THREE PHASE, 75KVA, WYE  
5% VOLTAGE REGULATION, 95% POWER FACTOR  
200AMP, 3 POLE, MAIN CB

**EQUIPMENT SCHEDULE**

**EQUIPMENT IDENTIFICATION NUMBER**

		BTU / HOUR	WEIGHT (LBS.)				
			DESCRIPTION	DIMENSIONS (IN.)			
				W	x	D	x
1	23920	TOSHIBA X-VISION CT GANTRY	31.90	66.90	69.30	2866	
2		TOSHIBA X-VISION CT PATIENT COUCH	94.10	24.40	15.20	728	
3	9219	TOSHIBA X-VISION CT CONTROL CONSOLE	33.50	51.20	45.30	552	
4	N/A	(OPTIONAL) MEDRAD MCT PLUS INJECTOR HEAD (CEILING MOUNTED)	26.00	26.00	60.25	30	
5	320	(OPTIONAL) MEDRAD MCT 305 CONTROL CONSOLE (COUNTER MOUNTED)	19.00	12.59	10.50	43	
6		(OPTIONAL) MEDRAD MCT RPS 350 REMOTE PANEL (COUNTER MOUNTED)	16.53	3.70	5.38	5	
7	1000	IMATION DRYVIEW 8700 PLUS LASER IMAGER	26.00	32.00	52.00	550	

**TOSHIBA X-VISION**

**GAMMA TECH, INC**  
 129 Wild Berry Lane  
 Hampstead, NC 28443  
 Ph (910) 270-9787  
 Fax (910) 270-9767  
 www.gammatech.net

© PLANNING DIV.

FILE NAME:  
TOSHIBA X VISION  
 REVISION DATE:  
01.06.2012